

# THE LP SHIELD

*Keep Your Package Dry!*

## USER MANUAL

①




LP SHIELD is made to the highest quality standard for comfort & durability

②




turn product 50% inside out so white interior is exposed

③

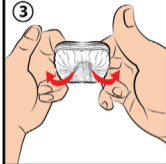


Hold it tightly with your hand and curl your fingers up tightly inside out

④

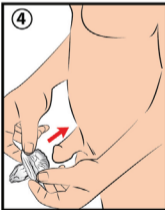


While holding LP Shield open with both hands fully insert head of penis into tip of device.

⑤

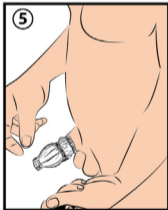

Roll product back towards body to fully apply product

⑥

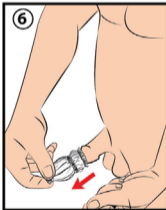

To remove product simply pinch at tip of product and pull away from penis.

**IF NOT OVERLY SOILED PRODUCT MAY BE RE-USED 3-4 TIMES BEFORE DISCARDING.  
"WHEN DISCARDING SOILED PRODUCT, THROW IN TRASH ("DO NOT FLUSH")**